

# Disposable Half Mask

3-layer disposable half mask Type II, manufactured in compliance with the Italian Law in force art 15 comma 2 of Decree Law n.18 of the 17 of March 2020. It cannot be used after the end of the epidemiological pandemic without the manufacturer's authorization. Authorised by the Italian National Institute of Health, Aut. COV1121.

**10 pieces per box, one size only.**

## COMPOSITION:

First layer: 25 g/sqm Polypropylene (PP) non-woven spunbond fabric

Second layer: 25 g/sqm Polypropylene (PP) meltblown electrostatic filter

Third layer: 25 g/sqm Polypropylene (PP) non-woven spunbond fabric

Filtering power BFE >98% (**actual value certified in Italy: 99%**)

Breathability <40 Pa/cm<sup>2</sup> (**actual value certified in Italy: <22,5 Pa/cm<sup>2</sup>**)

## INSTRUCTION FOR USE:



1. Using a hand sanitiser carefully disinfect your hands. In alternative, you can wash your hands for 60 seconds. Open the ziplock and take out one of the masks. Close the pack again using the ziplock in order to preserve the masks left inside.
2. Wear the mask so that the colored part is on the outside and the nose clip is upwards. Adjust the ear straps.
3. Adjust the mask on your face in order to cover your nose and chin.
4. Using both hands adjust the nose clip to the bridge of your nose. Do not leave the half mask hanging from your neck.
5. Once the mask is on, do not touch it with your fingers.
6. Under any circumstance that may reduce the mask efficiency, take it off immediately and where needed, carry out all necessary medical checks.
7. Replace the mask if it is damaged, contaminated or wet.
8. After taking off the mask, sanitise your hands fully.
9. Dispose the mask correctly. Do not put it back in the packaging once it has been used. If used outside of a medical environment, throw it away in the mixed bin, close the bin bag and put it inside a new bin bag. In a healthcare environment dispose it as medical waste.
10. Maximum use: 4 consecutive hours.

## WARNINGS:

Disposable, non-sterile. Use it only when the environment is adequately ventilated and with enough oxygen supply. Do not use if you are not aware of the type of contaminants and their percentage in the environment. This mask does not protect against gas or fumes. Do not alter, modify or clean the mask. Please take it off if the breathing resistance is excessive. Do not use beyond the expiry date. Store it in a dry place with a temperature range between -10° and 40°C; maximum humidity: 80%. When used jointly with other devices no risks or interference have been reported.

**DATE OF PRODUCTION:** see on packaging **EXPIRY DATE:** see on packaging

**PRODUCT CODE:** SMIDPI01F **BATCH:** see on packaging **MADE IN CHINA**

Imported and distributed by Polonord Adeste Srl  
Via C. Bonazzi 7, 40013 Castel Maggiore (BO), Italy  
Company with UNI EN ISO 9001:2015 quality management system

